FICHA TÉCNICA

N07-R11

REV.02

09/11/2020

Pag. 1 de 9

PT

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Pag. 7 de 9

EN

1. IDENTIFICATION OF PRODUCT AND COMPANY

PRODUCT

BIOLOGICAL TEST

Biological indicator to monitor steam sterilization cycles

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2. PRODUCT PRESENTATION

PRODUCT

Biological Indicator - Result in 24 hours.

PRESENTATION

Pack with 10, 50 and 100 units.

3. CHARACTERISTIC AND INFOMATIONS

CHARACTERISTICS

Self-contained biological indicators with 24-hour results, allows monitoring the effectiveness of steam sterilization.

It consists of a strip of filter paper with spores of *Geobacillus Stearothermophilus* and an ampoule with culture medium and pH indicator, inside a plastic bottle with a label marked with a chemical indicator for exposure control.

Easily identifies with a brown cap.

Requires specific incubation conditions from 55 to 60 ° C for 24 hours to obtain the final result.

System tested and approved nationally and internationally.

FREQUANCY AND INDICATIONS OF USE

Biological Test indicators must be placed in an appropriate test package, following the frequency defined in accordance with local legislation (ANVISA, FDA, etc.). The number of test packages must be proportional to the size of the autoclave.

Indicated to monitor steam sterilization cycles at temperatures between 121 and 134 °C.

INSTRUCTIONS FOR USE

1. Identify each Biological Test Biological Test by writing the equipment and load identification numbers, as well as the processing date on the ampule label. Place the Biological Test in a suitable test package.
2. Place the test package in a location that is difficult to access inside the autoclave, usually in the lower portion, close to the door, over the drain or according to the autoclave manufacturer's instructions.
3. Expose the load in a normal sterilization cycle, according to the manufacturer's instructions. At the end of the cycle, remove the test pack and let it cool for at least 10 minutes before removing the Biological Test.

DATA SHEET

N07-R11

REV.02

09/11/2020

Pag. 8 de 9

4. Activate the Biological Test by compressing the flask until the glass ampoule is broken, so that the culture medium contacts the spores of the paper disc.
5. Place the activated Biological Test in an appropriate incubator for 24 hours at a temperature between 55 to 60 ° C.
6. Start the biological indicator incubation process at the latest two hours after the end of the cycle.
7. If the spores survived the sterilization cycle, the culture medium will turn yellow (positive test), indicating that the sterilization was inadequate, if destroyed, the medium will remain purple (original color).

Note: For best results, it is recommended to check the ampoules every 6 hours. Every sample must be registered, autoclaved and discarded. Record test results on the back and keep them on file.

SPECIFICATIONS AND COMPOSITION

Population = minimum of 10^5
D121 value = 1.5 to 3 minutes
D134 value = 0.2 to 0.4 minutes

Polypropylene tube: length 52.0 mm x outer diameter 8.70 mm x wall thickness 0.50 mm.
Polypropylene cover: length 14.0 mm x outer diameter 10.40 mm x wall thickness 0.80 mm.
Spore carrier: filter paper 6.0 mm in diameter, 250 g / m².
Glass ampoule: length 43.0 mm x outer diameter 6.5 mm x wall thickness 0.5 mm.
Culture Medium: 0.5 - 0.7 mL.
Cover filter: 10 mm in diameter, 140 g / m².
Bacterium: *Geobacillus Stereothermophilus*.

STORAGE

Biological Test biological indicators should be stored in the sun at a temperature between 15 and 30 ° C and relative humidity between 35 and 60%. They should not be stored near chemical sterilizers.
Under correct storage conditions, Biological Test indicators are valid for 24 months after the date of manufacture.

APPLIED REGULATIONS

ANSI/AAMI/ISO 11138-1:2017, **Sterilization of health care products Biological Indicators** Part 1: General requirements.
EN 866-3:2000, **Biological Systems for resting sterilizers and sterilization processes** Part 3: particular systems for use moist heat sterilizers.

4. TRANSPORT INFORMATION

RISK CLASSIFICATION

This product is not classified as dangerous according to ANTT resolution No. 420 of 12 February 2004.

DATA SHEET

N07-R11

REV.02

09/11/2020

Pag. 9 de 9

5. PICTURES OF PRODUCT