FICHA TÉCNICA

N07-R11

REV.01

09/11/2020

Pag. 1 de 6

PT

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Pag. 5 de 6

EN

1. IDENTIFICATION OF PRODUCT AND COMPANY

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3. CHARACTERISTIC AND INFOMATIONS

CHARACTERISTICS

The 2i The 2i mini incubator is made up of 6 cavities for simultaneous incubation of up to 6 biological indicators in the temperature range from 55 to 60 ° C. It has an exclusive system to break ampoules and a translucent protective cap, bringing more safety and hygiene during incubation. Its incubation chamber is made of aluminum. Automatic bivolt, no need to change the voltage switch.

FREQUANCY AND INDICATIONS OF USE

Indicated for incubation of biological indicators between 55 to 60 ° C.

INSTRUCTIONS FOR USE

1. Plug the power cord into the outlet (110-220V). At this moment, the indicative LED will light up, indicating that the equipment is on.
2. After 15 minutes of heating the 2i mini incubator is ready to be used. After this warm-up period, the self-contained biological indicators can be incubated.
3. Open the cover and insert the biological indicators. To carry out microbiological monitoring, follow the instructions for use of the manufacturer of biological indicators.

SPECIFICATIONS AND COMPOSITION

Incubation capacity: up to 6 indicators
Incubation temperature: 55 to 60 ° C
Incubation chamber composition: aluminum
Fairing composition: ABS plastic
Voltage: Bivolt
Power: 10 watts
Frequency: 50/60 Hz Guillotine blade: # 12 scalpel.

DATA SHEET

N07-R11

REV.01

09/11/2020

Pag. 6 de 6

STORAGE

Store product in a dry place, away from sunlight.
Never touch the incubation chamber. This region is heated and can cause burns.

APPLIED REGULATIONS

ANSI/AAMI/ISO 11138-1:2017, **Sterilization of health care products Biological Indicators** Part 1: General requirements.

EN 866-3:2000, **Biological Systems for resting sterilizers and sterilization processes** Part 3: particular systems for use moist heat sterilizers.

4. TRANSPORT INFORMATION

RISK CLASSIFICATION

This product is not classified as dangerous according to ANTT resolution No. 420 of 12 February 2004.

5. PICTURES OF PRODUCT