FICHA TÉCNICA

N07-R11

REV.02

22/02/2021

Pag. 1 de 7

PT

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Pag. 6 de 7

EN

1. IDENTIFICATION OF PRODUCT AND COMPANY

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3. CHARACTERISTIC AND INFOMATIONS

CHARACTERISTICS

Indicated for internally monitoring steam sterilization packages. When exposed to the sterilization process, they undergo chemical and physical changes, such as color changes. This is so that the professional has greater quality control of the sterilization process through practical monitoring, but among the most effective.

The class 5 chemical indicator allows the following parameters to be read: time, steam and temperature.

Printed with indicator ink, which changes from pink to black when subjected to steam sterilization.

Immediate result, immediately after sterilization.

Clear, simple and straightforward identification and registration system.

FREQUANCY AND INDICATIONS OF USE

The class 5 indicator is designed to be used in steam autoclaves that are sterilized in cycles of 121 ° C for 17 minutes or 134 ° C for 3.5 minutes.

The use of chemical indicators is recommended in all containers that undergo sterilization and in all cycles

INSTRUCTIONS FOR USE

Place an indicator inside each package, bag, or tray to be steam sterilized.

In rigid containers, place an indicator at each corner, or at least two diagonal or opposite corners, to ensure steam reaches inaccessible areas.

After completing the process, analyze the results. The indicator to change the reference color to consider that the sterilization conditions have been reached.

N07-R11

REV.02

22/02/2021

Pag. 7 de 7

DATA SHEET

SPECIFICATIONS AND COMPOSITION

Initial color: pink
Final color: brown
Indicator ink
Length: 103mm
Width: 16mm
Free of heavy metals.

STORAGE

Keep the indicators in a dry place, protected from light and at a temperature of 2 to 28 ° C. Do not get wet. Do not store near sterilizing agents.

Indicators are valid for 2 years from the date of manufacture when stored under proper conditions. The expiration date and batch of the product are specified on the packaging label of each production unit. After expiration, discard any unused indicators.

APPLIED REGULATIONS

RDC 16: 2013 **Good manufacturing practices for medical products and diagnostic products for in vitro use.**

ISO 9001: 2015 **Quality management systems.**

ISO 13485: 2016 **Medical devices - Quality management systems - Requirements for regulatory purposes.**

ISO 11140: 2019 **Sterilization of medical devices - Chemical indicators. Part 1: General requirements.**

4. TRANSPORT INFORMATION

RISK CLASSIFICATION

Observe the storage conditions. Transport in closed and reinforced boxes to avoid damage. The transport of this product does not present any risk to human health

5. PICTURES OF PRODUCT